
www.bienair.com
 Other addresses available at
 dental@bienair.com
 Tel. +41 (0)32 344 64 64 Fax +41 (0)32 344 64 91
 Langgasse 60 Case Postale 2500 Biel/Bienne 6 Switzerland

4VR 400 B4VR 400

 Ne pas stériliser / Do not sterilise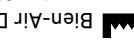

FRA	Mode d'emploi
DEU	Anleitung
ENG	Instruction
ESP	Instrucciones de uso
ITA	Istruzioni
RUS	Инструкция по эксплуатации
JPN	取扱説明書

REF 2100037-0000/2019.10

CE
0123

a Assortiment livré / Set supplied

4VR 400

REF 1600397-001 / REF 1600397-0


B4VR 400

REF 1600398-001 / REF 1600398-0

b Accessoires en option / Optional accessories

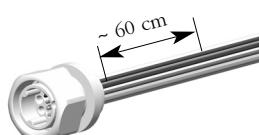REF 16000280-001

REF 1300367-010

REF 1300368-010

REF 1600036-006

6x

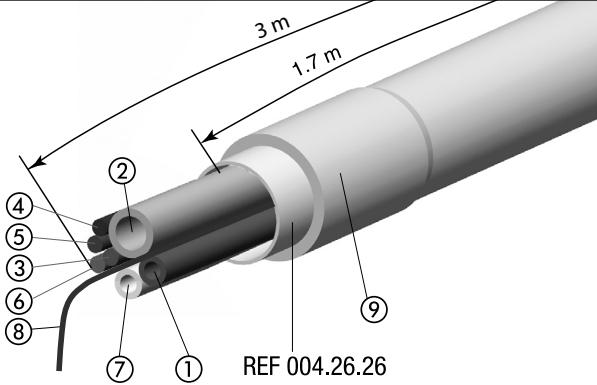
fig. 1

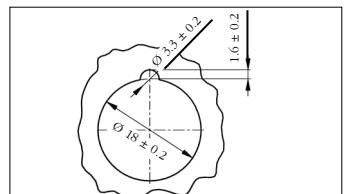
fig. 4

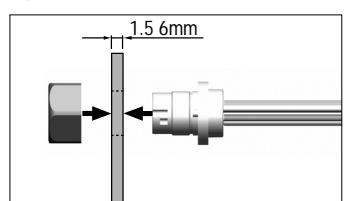
fig. 5

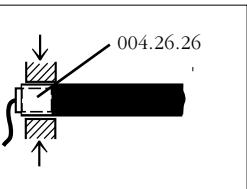
fig. 2

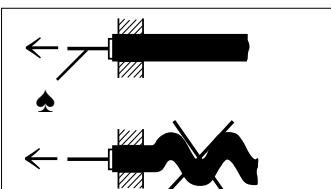
fig. 3

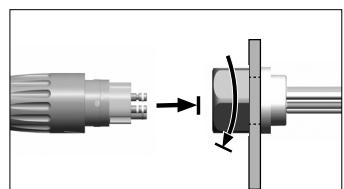
fig. 6

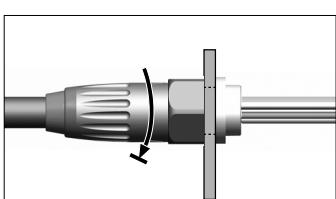
fig. 7

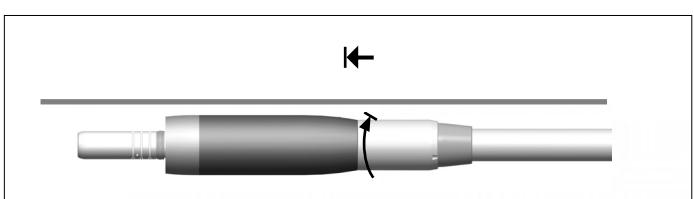
fig. 10

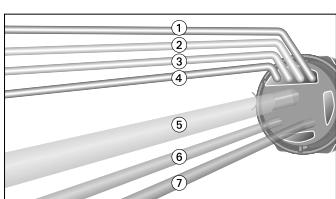
fig. 8

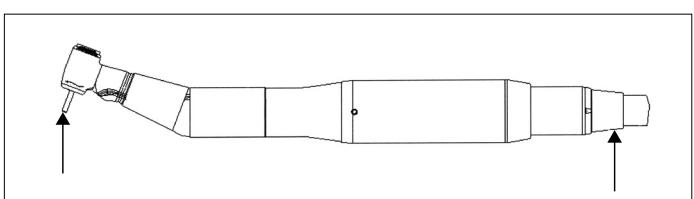
fig. 11

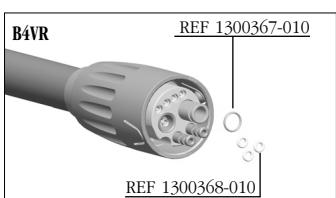
fig. 9

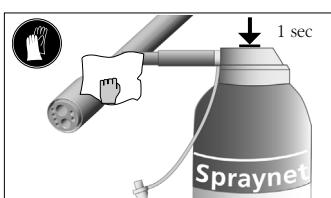
fig. 12

BienAir®
Dental

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English

Type

New hose with short, rotating coupling, ensuring easy use of the MC3 micromotor and its handpiece. For MC3 motors with and without light. Coupling rotates through 400°. Hose Ø 10 mm, standard length 1.7 m or special length up to 3 m with price supplement.

Intended use

Product intended for professional use only. Use in dentistry for prophylaxis, general dentistry and endodontic work.

Technical data and assembly

Classification

Class IIa, «active medical devices», as per section III «Classification», Rule 2 of Annex IX to European Directive 93/42/EEC, namely: «Non-invasive devices intended for channelling liquids or gases for the purpose of administration into the body, which may be connected to an active

medical device in Class IIa».

These hoses are not suitable for pressure higher than 5 bar (500 kPa, 72 psi).

Standard length

Weight 144 g

Maximum direct current

2.5 A (5 A 10 s)

Sustained voltage according

to CEI 60601-1

500 Vac/1 min

Flow rate of the air-intake tube at 2.5 bar (36.28 psi)

20 normliter/min

Flow rate of the air-spray tube at 2.5 bar (36.28 psi)

12 normliter/min

Flow rate of the water-spray tube at 2 bar (29 psi) 3 0 0

ml/min

Maximum total length of use

3 m

Assembly 4VR 400 fig. 1

Preserve the initial alignment of the

wires and tubes. Place the metal reinforcement ring REF 004.26.26 in the chucking zone **fig. 2**.

Slightly pull the security string **fig. 3**. Resistance to traction: 60 N maximum.

Description fig. 1

- 1 Ø 1.5/2.5 mm green: waterspray
- 2 Ø 1.5/2.5 mm white: airspray
- 3 (+) red: motor
- 4 (0V) black: motor
- 5 (+) brown: bulb
- 6 (0V) blue: bulb
- 7 Ø 2.8/4.1 mm white: cooling MC3
- 8 security string
- 9 sheath silicone

Assembly B4VR 400 fig. 4 to 9

Installing the socket in the unit

Drilling for the wall Ø 18 mm ± 0.2
Machine a countersink for the catch **fig. 4**.

Nut's gripping 150 Ncm maximum
Minimum wall thickness 1.5 mm
Maximum wall thickness 6 mm

Installing the hose on the unit

The dot engraved on the rear cover of

the hose should be aligned with the contra-angle tool fitted on the MC3 motor **fig. 11**.

Description connector B only fig. 8

- 1 (OV) black motor
- 2 (OV) blue bulb
- 3 (+) brown bulb
- 4 (+) red motor
- 5 Ø 2.8/4.1 mm white Cooling
- 6 Ø 1.5/2.5 mm white airspray
- 7 Ø 1.5/2.5 mm green waterspray

Assembly on motor fig. 10

Electric micromotors MC3 + tube without nut.

Accessories

Quick connector on unit side only for Ø 18 mm hole, with rotation prevention notch. Standard length 60 cm.

Maintenance

Non-sterilisable

Cleaning

Use Spraynet from Bien-Air Dental **fig. 12**.

Spray the outside of the equipment

and carefully clean the surface.

Disinfection

Disinfect using a clean cloth soaked in a suitable product. Products containing acetone, chlorine and bleaches are not recommended as disinfectants. To keep the surface of the hose in good condition, it is advisable to periodically wipe the complete length of it with a cloth dusted with talcum powder.

Never immerse in disinfectant solutions.

Not suitable for ultrasonic bath.

Environment

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical

improvements to its equipment, without amending these instructions. For all additional information, please contact Bien-Air Dental SA at the address indicated on the back cover.

Generalities

The device must be used by a qualified person in accordance with the current legal provisions concerning industrial safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre that is approved by Bien-Air Dental;
- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and

third parties from any danger, and must avoid contamination through the use of the product.

The device is intended for medical treatment only; any use other than that for which this product is intended is unauthorised and may be dangerous. The medical device meets all the current legal requirements.

The device is not authorised for use in an explosive atmosphere (anaesthetic gas).

Recommendations

It is essential to use dry, purified compressed air in order to ensure the long working life of the device. Maintain the quality of the air and the water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

Rest the device on a suitable support to avoid risks of infection for yourself, the patient or third parties. Only use maintenance products and

components from Bien-Air Dental. The use of other products and components can void the guarantee.

Guarantee

Terms of guarantee

Bien-Air Dental grants the operator a guarantee covering all functional defects, material or production faults. The device is covered by this guarantee for 12 months from the date of invoicing.

Bien-Air Dental offers a 24-month guarantee for the glass-bar light conductors.

In the event of justified claim, Bien-Air Dental or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting

from:

- excessive wear and tear
- improper use
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the

air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its con-

sequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorised by Bien-Air Dental.

of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

Claims under the terms of the guarantee will be considered only on presentation, together with the product,

REF	Legend	List of Bien-Air Dental SA registered trade mark products ®:			
1600397-001	4VR 400, 4-hole coupling with electrical socket rotating through 400°. Hose Ø 10, grey silicon	Aquilon®	Eolia®	Lubrifluid®	Prestilina®
1600397-0	4VR 400, special length	Bora®	Gyro®	Lubrimed®	Spraynet®
1600398-001	B4VR 400, 4-hole coupling with electrical socket rotating through 400° and quick bayonet connector. Hose Ø 10, grey silicon	Boralina®	Gyrolina®	MX®	
1600398-0	B4VR 400, special length	ChiroPro®	Isolite®	PowerCare®	
104.44.02-1	Connector B only				
1600280-001	Quick connector on unit side only for Ø 18 mm hole, with rotation prevention notch. Standard length 60 cm				
1300367-010	O-ring Ø 3.2 x 0.6				
1300368-010	O-ring Ø 1.5 x 0.75				
1600036-006	Spraynet, cleaning spray 500 ml, box of 6 cans				


Symbols

Manufacturer.

CE Marking with number of the notified body.

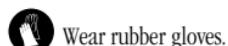Wear rubber gloves.

Move in the direction indicated.

Attention.

Move fully to the stop, in the direction indicated.

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